


HYALACE

BEST FOR YOUR SKIN

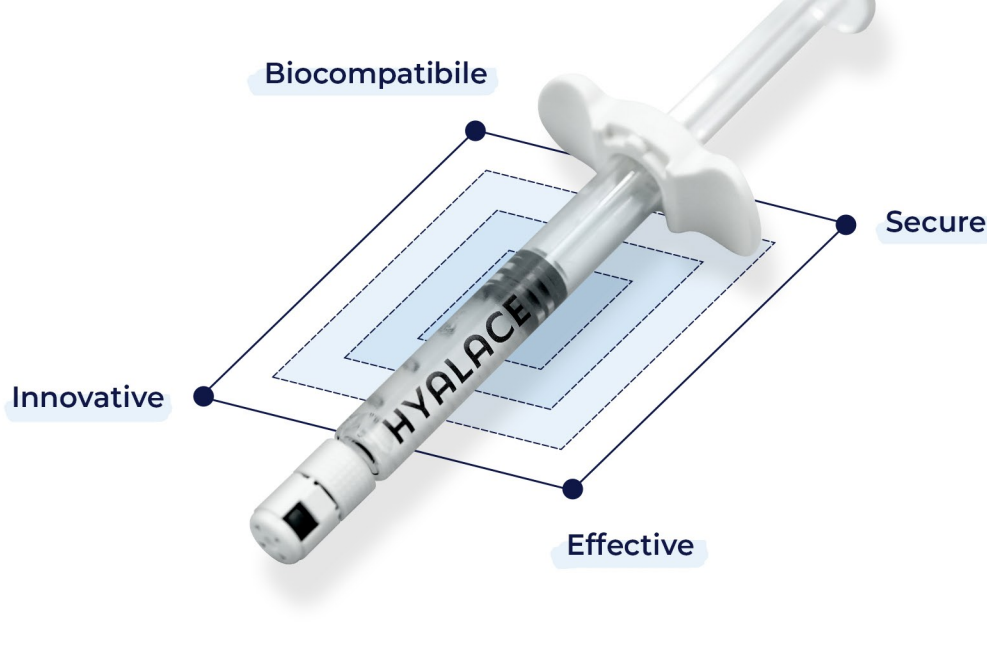


A new ally aging which nourishes the dermal cells and restores firmness of the skin



HYALACE is an injectable anti-ageing treatment made from hyaluronic acid for skin that is losing its elasticity and firmness with age. One of the highest concentrations of hyaluronic acid on the market 64mg/2ml.

Benefits of HYALACE

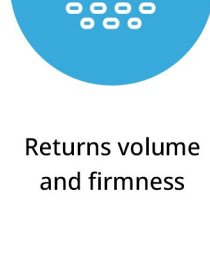


Returns volume and firmness



Improves skin structure




Deeply hydrates your skin

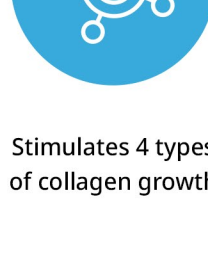

Stimulates 4 types of collagen growth

HYALACE areas of treatment

- Face
- Hands
- Neck
- Decollette

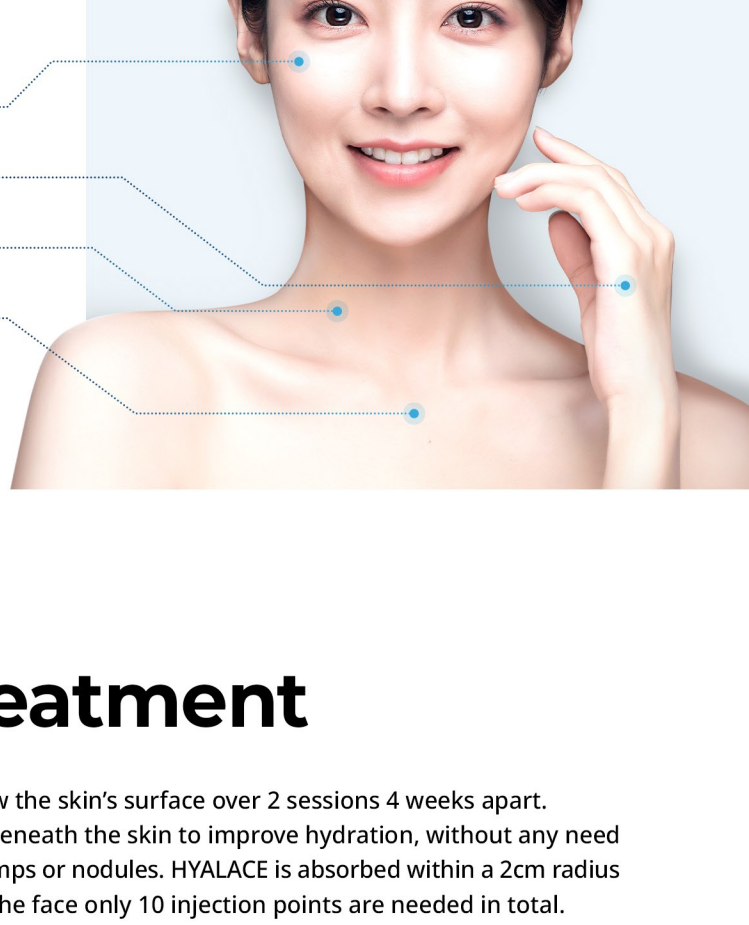


Treatment

HYALACE is injected just below the skin's surface over 2 sessions 4 weeks apart. After injection, it disperses quickly beneath the skin to improve hydration, without any need for massage or the risk of leaving lumps or nodules. HYALACE is absorbed within a 2cm radius of its injection site, so to treat the face only 10 injection points are needed in total. Other parts of your body such as your hands, neck, or décolleté may need a different amount.



5 Point BAP technique What is special about it?

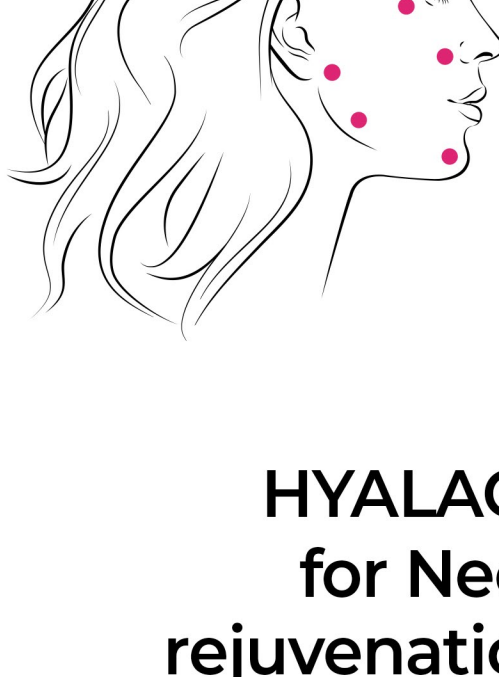


Unlike Skinboosters or Rejuran, **HYALACE only needs 5 points per cheek for treatment of the whole face!**

The 5 Bio Aesthetic Points (BAP) are specially selected areas of the face that have no large blood vessels or nerve branches, thereby minimizing the risk of bruising and improving safety while allowing for maximal diffusion of HYALACE's hybrid HA complex across the entire face. With only 5 injection point per cheek, discomfort and downtime is minimal to none. In total, there are 10 injection points for the entire face. Each treatment requires 2cc of HYALACE.

HYALACE for Neck rejuvenation

Neck HYALACE treatment is excellent for targeting wrinkled or sagging neck skin.

Neck anti-aging benefits are stimulated from deep within the skin. 10 points along the neck are specially chosen to avoid vital structures such as blood vessels. The treatment experience is similar to HYALACE for the face.

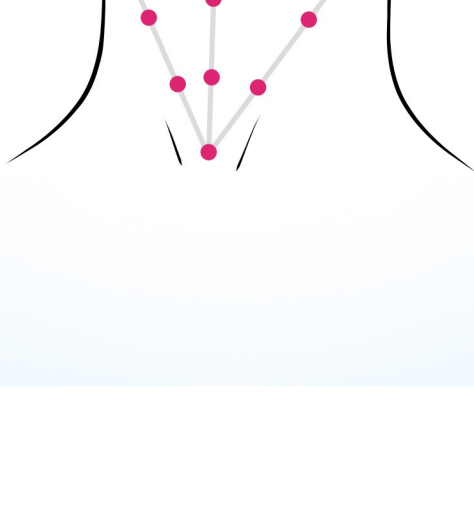

Duration of the effect

The effect generally lasts from 6 to 9 months depending on individual skin type factors such as age and lifestyle.

Caution and recommendation

- After approximately 30 minutes, you should be able to return to your normal 'light' activities, including driving.
- It is recommended that you avoid wearing make-up for 24 hours and that you take a day off from strenuous physical exercise, such as visiting the gym or playing your favorite sport.
- Avoid saunas or a steamy bath, as well as intense sun or UV light, for three to four days.

HYALACE rejuvenates the skin and increases collagen and elastin production in the different layers of the skin by

- Stimulating fibroblasts
- Supporting Keratinocyte viability and promoting wound healing
- Activating Keratinocytes
- Recruitment of stem cells in adipocytes and improving fat tissue renewal for youth full looking skin

Contraindications

Contraindications include patients with bleeding disorders, immunocompromised disorders and those with unrealistic expectations.

Specification

Product Name	HYALACE
Packing unit	2 ml * 1 Syringe, Instruction for use
Storage	Store at room temperature(1-25°C), protect from sunlight
Expiration date	24 months from date of manufacture. Check the box for the expiration date.
Ingredients	Water, Sodium Hyaluronate, Sodium Chloride

IMPORTANT : comes **WITHOUT A NEEDLE**. The needle must be purchased separately. (Recommended needle size: 27G / 30G)